Rapid Response

3-in-1 Spiked Drink Pen

INSTRUCTIONS

English

Français

SCROLL

ENGLISH

Read the entire product insert prior to performing the test. Do not use the test after the expiration date printed on the package.

- 1** Remove the test pen from the pouch. For best results, the assay should be performed within two hours.

- 2** Take the test pen out of the extraction tube. Keep the extraction tube aside.

- 3** Remove the protector from the test pen to reveal the sample tip.


- 4** **OPTION 1:** Dip the sample tip into the beverage.

- OPTION 2:** Add the beverage into the protector cap OR pour the beverage into a clean, dry container and then dip the swab tip into the collected sample.

Hold the sample tip in the beverage sample for 2 seconds. Do not dip past the blue ring.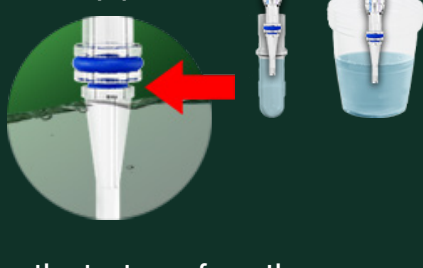

- 5** Remove the test pen from the sample. Insert it into the extraction tube until the middle of the test pen, supporting ring, aligns with the top edge of the tube. Ensure the test pen is secured in place. Place the test upright on a flat surface while the test is running.

RESULTS INTERPRETATION

For GHB, read the results at 3 minutes.

POSITIVE **NEGATIVE**

For BZO/KET, read the results at 5 minutes.

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For beverage drug testing use only. Not an IVD. This test is NOT intended to determine the purity, composition, or if the liquid being examined is safe to use. A positive or negative test result is NOT a guarantee that the liquid being tested is safe or unsafe to use. **Any tested liquid is consumed or used at the consumer's own risk.** Rapid Response™ 3-in-1 Spiked Drink Pen provides information to consumers but does not eliminate risks. **Users of Rapid Response™ 3-in-1 Spiked Drink Pen accept responsibility for any consequences, including injury or death, that may occur after consuming a tested liquid.** Many factors come into play when testing a liquid, including but not limited to mixture of multiple substances, solubility, and pH of the sample. Technical or procedural errors, as well as other substances, and factors may affect the accuracy of the Rapid Response™ 3-in-1 Spiked Drink Pen and can cause false results. A positive result indicates the presence of GHB, Ketamine, and/or Benzodiazepine only and does not indicate quantity or concentration. A negative result does not rule out the presence of GHB, Ketamine, and/or Benzodiazepine, or any other potentially harmful substances.

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© 2025 BTNX Inc. All rights reserved. Rapid Response™ is a trademark of BTNX Inc.