

[REF] COF-19CGHU1

[REF] COF-19CGHU2

[REF] COF-19CGHU4

**FDA Authorized COVID /  
 FLU A / FLU B Home Test**

# Rapid Response™ Influenza A&B + COVID-19 Antigen Detection Test (Anterior nasal)

**INFECTIOUS DISEASES**


The Rapid Response™ Influenza A&B + COVID-19 Antigen Detection Test is for the detection and differentiation of Influenza A, Influenza B, and SARS-CoV-2 antigens from anterior nasal swab specimens.

Save time with this simple and convenient at-home test that provides results in 15 minutes.

## Kit Content

Each kit contains the materials required to complete 1 (COF-19CGHU1), 2 (COF-19CGHU2), or 4 (COF-19CGHU4) tests. Each test requires:

- Sealed Test Cassette
- Sterile Nasal Swabs
- Prefilled Extraction Tube
- Extraction Tube Tip
- Tube Holders
- Package Insert
- Quick Reference Guide (QRG)

## Product Details

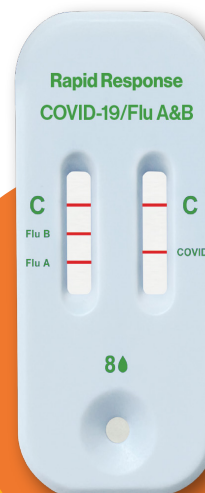
**Sample:** Anterior nasal swabs

**Format:** Test Device

**Time-to-Result:** 15 minutes

**Storage Condition:** 36-86°F/2-30°C

**Test Principle:** Lateral Flow Assay



### 3-in-1 Test

Quickly detects antigens from COVID-19, Influenza A and Influenza B at home.



### User-Friendly

The kit includes simple instructions and all the materials needed to run the test.



### Easy & Convenient

Get fast results in just 15 minutes from a single swab sample.



### For Adults & Children

Suitable for testing individuals 2+ years old.\*

For *in vitro* diagnostic use only.  
 OTC in U.S.

\* Swabbing should be performed by an adult for children aged 2 to 13. User must be aged 14+ to perform self-test.

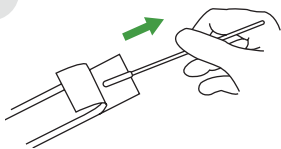
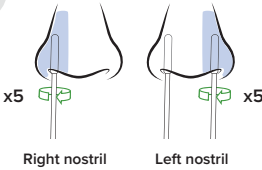
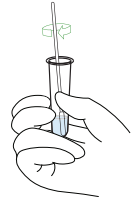
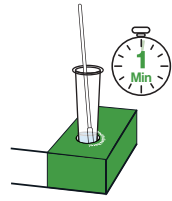
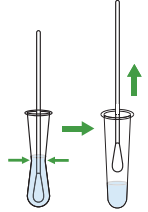
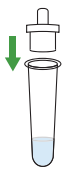
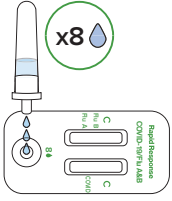

[REF] COF-19CGHU1

[REF] COF-19CGHU2

[REF] COF-19CGHU4

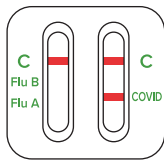
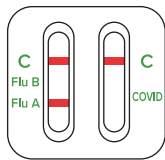
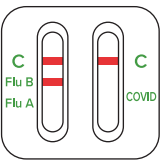
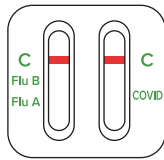
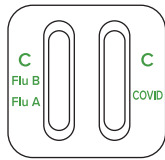
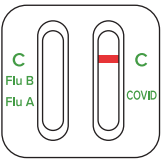
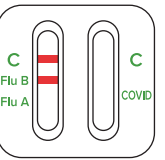
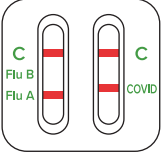
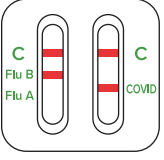
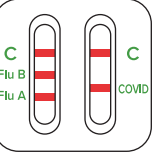
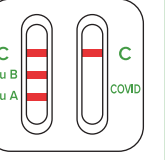
# Influenza A&B + COVID-19 Antigen Detection Test

## Testing Procedure

<p><b>1</b></p>  <p>Remove the swab from the pouch. Carefully insert the swab no more than 3/4 inch (1.5 cm) into the nostril.</p>	<p><b>2</b></p>  <p>Slowly rotate the swab at least <b>5 times</b> against the nostril wall for at least 15 seconds. Remove the swab and repeat in the other nostril using the same swab.</p>	<p><b>3</b></p>  <p>Immerse the swab into the prefilled extraction tube and swirl the swab in the buffer. Mix the sample thoroughly by making at least <b>6 circles</b>.</p>	<p><b>4</b></p>  <p>Start the timer. Leave the swab in the extraction tube for 1 minute.</p>
<p><b>5</b></p>  <p>After 1 minute, pinch the tip of the swab from the outside of the tube to remove any excess sample in the swab. Remove and discard the swab.</p>	<p><b>6</b></p>  <p>Hold the tube upright and insert the extraction tube tip into the tube opening. Ensure a tight fit to prevent leaking.</p>	<p><b>7</b></p>  <p>Invert the extraction tube and squeeze <b>8 drops</b> of test sample into the sample well. Then discard the tube.</p>	<p><b>8</b></p>  <p>Start the timer. Read the results at 15 minutes. Results may be invalid before 15 minutes or after 20 minutes.</p>

These instructions are for illustrative purposes only. Please read the instructions supplied with the test before use.

## Result Interpretation

POSITIVE				NEGATIVE		INVALID		
								
COVID-19 POSITIVE	FLU A POSITIVE	FLU B POSITIVE						
								
COVID-19 + FLU A POSITIVE	COVID-19 + FLU B POSITIVE	COVID-19 + FLU A + FLU B POSITIVE	FLU A + FLU B POSITIVE					

Product Code	Product Name	Contents
COF-19CGHU4	Rapid Response™ Influenza A&B + COVID-19 Antigen Detection Test	4 Tests/Kit
COF-19CGHU2	Rapid Response™ Influenza A&B + COVID-19 Antigen Detection Test	2 Tests/Kit
COF-19CGHU1	Rapid Response™ Influenza A&B + COVID-19 Antigen Detection Test	1 Test/Kit

To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at [info@lochnessmedical.com](mailto:info@lochnessmedical.com)