

REF FYL-18S7-100, FYL-18S7-10


Rapid Response™ Fentanyl Test Strip

(Liquid / Powder)

Fentanyl is a synthetic opioid related to the phenylpiperidines that is commonly used for moderate to severe pain relief. Continued use of fentanyl can lead to opioid use disorder.

The Rapid Response™ Fentanyl Test Strip (Liquid / Powder) is an easy to use, cost-effective dipstick test. It is a rapid, qualitative screening test that quickly and reliably detects Fentanyl at cut-off levels above 200 ng/mL. This drug test can provide accurate and fast results as soon as 1 minute. For forensic use only. Not an IVD.

Testing Procedure

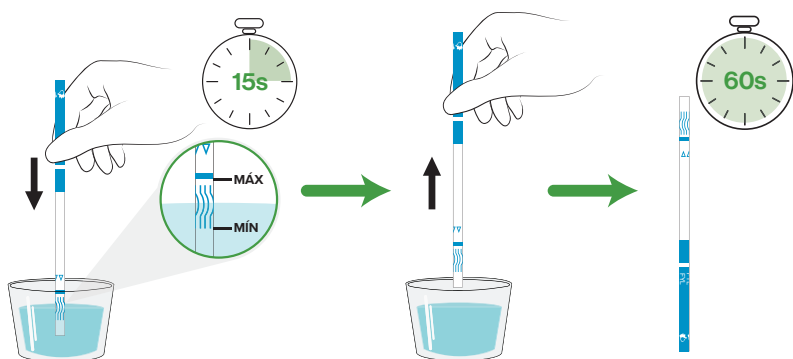


For illustrative purposes only. Refer to product insert for more details.

Results Interpretation


Positive
Contains
FYL



Negative
FYL could not
be detected

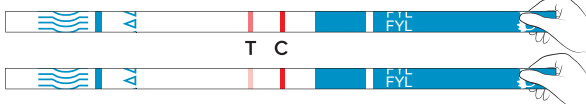

Invalid

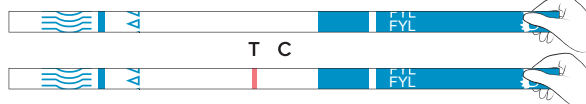

Kit Contents

- Individually packaged test strips
- Product insert
- Results interpretation card

Product Details

- Detects Fentanyl in concentrations at and above 200 ng/mL
- Results interpretation printed on every pouch
- Results in 60 seconds*
- Compact format for easy distribution

Detects Fentanyl Analogues including:

Carfentanil	Butyryl Fentanyl	p-Fluoro Fentanyl
Acetyl Fentanyl	Fentanyl	Furanyl Fentanyl
Valeryl Fentanyl	Ocfentanil	3-Methyl Fentanyl
Remifentanil	Sufentanil	

*A negative result can be interpreted as soon as both the test (T) and control (C) lines appear. A positive result can be interpreted once the control line has appeared and the background has cleared to white, typically by 60 seconds. Do not read results after 10 minutes.



bit.ly/3NHprTH

John Hopkins Research

Rapid Response™ Fentanyl Test Strips showed the lowest detection limit and highest accuracy among other fentanyl detection methods when tested by John Hopkins Bloomberg School of Public Health.



bit.ly/3qdshcC

Visit Our Website for more Harm Reduction Resources

To learn more about Rapid Response™ Fentanyl Test Strip (Liquid / Powder), scan this QR code.

Ordering Information

Product Code	Product Name	Contents
FYL-18S7-100	Rapid Response™ Fentanyl Test Strip (Liquid / Powder)	100 Tests
FYL-18S7-10	Rapid Response™ Fentanyl Test Strip (Liquid / Powder)	10 Tests

To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at **info@lochnessmedical.com**

Our Harm Reduction Line Up



Rapid Response™ Xylazine Test Strips

Product Code	Product Name	Contents
XYL-18S2-100	Rapid Response™ Xylazine Test Strip (Liquid / Powder)	100 Tests
XYL-18S2-10	Rapid Response™ Xylazine Test Strip (Liquid / Powder)	10 Tests
XYL-18S2-5K	Rapid Response™ Xylazine Test Strip Kit (Liquid / Powder)	5 Tests



Rapid Response™ Fentanyl Test Strips

Product Code	Product Name	Contents
FYL-18S7-100	Rapid Response™ Fentanyl Test Strip (Liquid / Powder)	100 Tests
FYL-18S7-10	Rapid Response™ Fentanyl Test Strip (Liquid / Powder)	10 Tests
FYL-18S7-5	Rapid Response™ Fentanyl Test Strip Kit (Liquid / Powder)	5 Tests



Rapid Response™ Benzodiazepine Test Strips

Product Code	Product Name	Contents
BZO-18S3-100	Rapid Response™ Benzodiazepine Test Strip (Liquid / Powder)	100 Tests



Rapid Response™ Nitazene Test Strips

Product Code	Product Name	Contents
NTZ-18S26-100	Rapid Response™ Nitazene Test Strip (Liquid / Powder)	100 Tests



Visit Our Harm Reduction Page

www.lochnessmedical.com/category/Harm-Reduction

To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at info@lochnessmedical.com