



REF DX.X-1V

Rapid Response™ Multi-Drug One Step Cup

(Urine)

DRUG SCREENING

The Rapid Response™ One Step Cup is an easy-to-use, one-step drug screening solution that can simultaneously detect multiple drugs in urine samples. This format is a great alternative to the industry's complicated testing processes and is available with and without adulterant tests.



Kit Contents

- Individually packaged test cups with integrated drug test panels and caps
- Adulteration color chart (when applicable)
- Product insert
- Security seals

Product Details

Sample: Urine

Format: Cup

Time-to-Result: 5 minutes

Storage Condition: 36-86°F/2-30°C

Test Principle: Lateral Flow Assay

Features

- Results in just 5 minutes
- No direct contact with the sample required
- Available with a wide range of parameters and adulteration test combinations
- Integrated procedural controls for each parameter



Leak-proof cap ensures no mess.

Peelable label ensures confidentiality and privacy.

READ	90	2	4	6	8	100
GREEN						
COLOR	32	3	4	6	7	38sec


Integrated temperature strip helps identify potential sample adulteration.


REF DX.X-1V


Multi-Drug One Step Cup

Testing Procedure


- 1**




Unscrew the cap from the cup.
- 2**

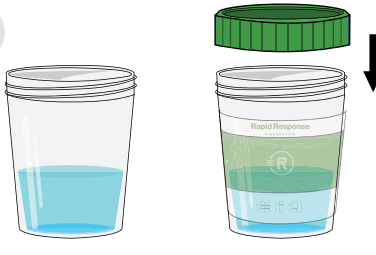

Collect the sample in the cup, then screw the cap onto the cup.
- 3**



Immediately start the timer for 5 minutes.
- 4**

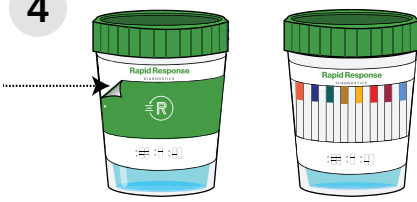


After 5 minutes, peel the sticker off and read the results.

These instructions are for illustrative purposes only. Please read the instructions supplied with the test before use.


Result Interpretation

Positive/Non-Negative

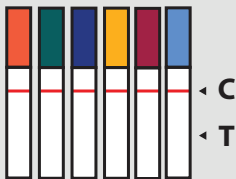


Only one colored band appears in the control region (C). No apparent colored band appears in the test region (T).

Negative

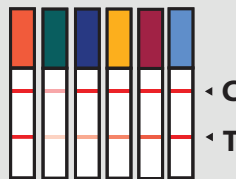

Two colored bands appear on the membrane. One band appears in the control region (C) and another appears in the test region (T).

Invalid

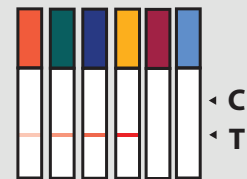

No line appears in the control region (C). The result is invalid.

Ordering Information

Product Code	Product Name	Contents
DX.X-1V, DX.X-1VA	Multi-Drug One Step Cup	100 Tests

To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at info@lochnessmedical.com


REF DX.X-1V

Multi-Drug One Step Cup

Popular Multi-Drug One Step Cup Products

Product Number	Product Name	Parameters
* D15.5-1VA	15-Panel Urine Drug Screen + Adulteration Test	AMP500 BAR300 BUP10 BZO300 COC150 ETG500 FYL10 MDMA500 MET500 MOP300 MTD300 OXY100 PCP25 K2-50 THC50 Cre/Sg/pH/Oxi
D14.5-1V	14-Panel Urine Drug Screen	AMP1000 BAR300 BUP10 BZO300 COC300 MET1000 MDMA500 OPI2000 MTD300 OXY100 PCP25 THC50 PPX300 TCA1000
* D12.26-1VA	12-Panel Urine Drug Screen + Adulteration Test	AMP1000 BUP10 BZO300 COC300 EDDP100 ETG500 FYL10 MDMA500 MET1000 MOP300 OXY100 THC50 + Cre/SG/pH
D12.11-1VA	12-Panel Urine Drug Screen + Adulteration Test	MOP300 OXY100 BZO300 OPI2000 MET1000 AMP1000 COC300 THC50 MDMA500 BUP10 BAR300 MTD300 + Cre/pH/Oxi
D12.5-1V	12-Panel Urine Drug Screen	AMP1000 BAR300 BUP10 BZO300 COC300 MDMA500 MTD300 MET1000 OPI2000 OXY100 THC50 PCP25
D10.32-1V	10-Panel Urine Drug Screen	COC300 MET1000 AMP1000 OPI2000 OXY100 MTD300 PCP25 BAR300 BZO300 MDMA500
D10.14-1V	10-Panel Urine Drug Screen	COC300 AMP1000 MET1000 THC50 MTD300 OPI2000 OXY100 PCP25 BAR300 BZO300
D10.3-1V	10-Panel Urine Drug Screen	COC300 AMP1000 MET1000 THC50 OPI2000 PCP25 BAR300 BZO300 MTD300 MDMA500
D5.75-1V	5-Panel Urine Drug Screen	COC300 MET1000 AMP1000 OPI2000 OXY100
D5.1-1V	5-Panel Urine Drug Screen	COC300 AMP1000 MET1000 THC50 OPI2000

* For forensic use only.

Available Drug Parameters				Available Adulteration Tests
<ul style="list-style-type: none"> • 6-MAM (Heroin) • Amphetamines • Barbiturates • Buprenorphine • Benzodiazepines • Cocaine 	<ul style="list-style-type: none"> • Ecstasy (MDMA) • ETG (Alcohol) • Fentanyl • Ketamine • THC (Marijuana) • EDDP (Methadone) 	<ul style="list-style-type: none"> • Methamphetamine • Morphine / Opiates • Oxycodone • Phencyclidine • Propoxyphene • Hydromorphone 	<ul style="list-style-type: none"> • Tricyclic Antidepressants • Tramadol • K2 • And more 	<ul style="list-style-type: none"> • Creatinine • Glutaraldehyde • Nitrites • Oxidants • pH • Specific Gravity

Ordering Information

Product Code	Product Name	Contents
DX.X-1V, DX.X-1VA	Multi-Drug One Step Cup	100 Tests

To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at **info@lochnessmedical.com**