Rapid Response®

Digital Pregnancy Test


Product Insert

Before Using

The Rapid Response® Digital Pregnancy Test is a rapid home test to aid in the early detection of pregnancy intended for home use. The device is a chromatographic immunoassay for the qualitative detection of human chorionic gonadotropin (hCG) in urine. This test can help detect pregnancy as early as 6 days before the day of the missed period (5 days before the day of the expected period). **Read these instructions carefully before taking a test.**

1 Take the Test

If you are testing before your expected period, use your first morning urine, as it contains the most pregnancy hormone. If testing on the day of or any day after your expected period, you can test any time of the day. You can test directly in your urine stream (Option A), or by collecting your urine in a clean, dry container (Option B).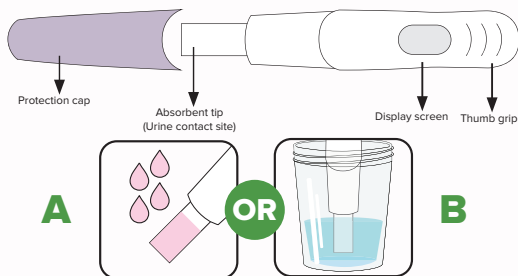

Do not touch the absorbent tip!

- After reading the instructions when you are ready to use the test, take the test out of the protective foil and remove the protective cap (do not discard). **If the display screen shows a '?' that means the test is no longer usable.**


Option A: Test directly in urine stream: Hold the test by the thumb grip with the absorbent tip pointed down and the display screen facing away from you while you pee. Urine should only touch the absorbent tip, do not urinate on the display screen. Place the tip in the urine stream for 5-10 seconds.

Option B: Test in collected urine sample: Collect fresh urine in a clean, dry container. Collect enough urine to submerge the entire absorbent tip, but do not submerge the plastic. Hold in the collected urine for 5-10 seconds.

- After sampling, the timer icon will be present on the display screen and start to flash after a few seconds. Immediately replace the cap over the absorbent tip. Lay the test on a flat, non-absorbent surface with the display window facing up.

2 Wait for Results

Wait 3 minutes for the test to complete. The flashing timer icon shows the test is in progress. Read your result after 3 minutes. The results should be displayed on screen.

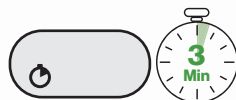3 Reading The Results

PREGNANT

PREGNANT (Positive)

Your result is 'Pregnant'. See your doctor to confirm and discuss next steps (see Question 5 and 6).

NOT PREGNANT

NOT PREGNANT (Negative)

Your result is 'Not Pregnant' (see Question 7).

?

INVALID

An error has occurred during testing. Discard the test, review the procedure and test again.

Questions

1. How does the test work?

Rapid Response® Digital Pregnancy Test detects a hormone in your urine that your body produces during pregnancy (hCG - human Chorionic Gonadotropin). The amount of this hormone in your body increases as pregnancy progresses.

2. How soon after I suspect that I am pregnant can I take the test?

You can take the test up to 6 days before your missed period (5 days before the day of your expected period). If you test before your expected period and get a "Not Pregnant" result, there is still a chance that you may be pregnant. See the chart below for results from clinical testing of the Rapid Response® Digital Pregnancy Test with early pregnancy samples.

Testing Early

Number of days before the expected period	0	1	2	3	4	5
% of samples from pregnant women giving a "Pregnant" result	>99%	>99%	>99%	94%	89%	72%

3. Do I have to test with my first morning urine?

If you are testing before your expected period, we recommend that you test with your first morning urine as it is usually the most concentrated of the day and would have the most pregnancy hormone (hCG) in it. If you are testing on the day of, or after your missed period, you can test at any time of the day. Avoid excessive fluid intake before testing.

4. Can other medical conditions affect the results?

Medications containing hCG can affect the result. Ectopic pregnancy and recent pregnancy even if not carried to full term can cause misleading results. If you are in or approaching the menopause you may obtain a false pregnant result even though you are not pregnant. These conditions must be ruled out before the test can be used to diagnose pregnancy. If you get unexpected results discuss them with your doctor.

5. What should I do if the test shows I am pregnant?

See your doctor to confirm that you are pregnant and to discuss the next steps you should take.

6. What if the test result shows I am pregnant but I think I am not?

If you received a "Pregnant" result but believe that you are not pregnant, schedule an appointment with your doctor. You may have a health condition, not pregnant, that is raising your hCG level.

7. The tests result shows I am not pregnant but I think I might be. What should I do?

Your hCG levels may not be developed enough to be detected yet. It may also be that you are experiencing an irregular menstrual cycle and have not had your period yet.

- If you tested early, you should test again in 3 days with another Rapid Response® Digital Pregnancy Test.
- You may not be pregnant. If you do not get your period within 7 days, you should retest with another Rapid Response® Digital Pregnancy Test. If this second test also indicates 'Not Pregnant' and you still have not had your period, see your doctor.

8. I used the test, but no result appeared on the display. What should I do?

If no result appears after 3 minutes but a "?" symbol or the display goes blank, an error has occurred during testing. Try again with a new test.

REF HCG-1DMNH25 REF HCG-1DMNH5 REF HCG-1DMNH3 REF HCG-1DMNH2 REF HCG-1DMNH1

Performance

Accuracy: Clinical evaluation was conducted comparing the Rapid Response® Digital Pregnancy Test to a market leading Pregnancy Test. The clinical trial included 180 urine samples: both assays identified 54 positive and 126 negative results. The results demonstrated >99% overall accuracy.

Sensitivity: The Rapid Response® Digital Pregnancy Test detects hCG in urine at a concentration of 10 mIU/mL or greater. The test has been standardized to the W.H.O. International Standard. The addition of LH (1,000 mIU/mL), FSH (1,000 mIU/mL), and TSH (1,000 µIU/mL) to negative (0 mIU/mL hCG, 3 mIU/mL hCG) and positive (10 mIU/mL hCG) samples showed no false positives.

Reproducibility: The Rapid Response® Digital Pregnancy Test has consistent performance and good reproducibility when tested by different operators using multiple test lots at different days.

Warnings

For *in vitro* diagnostic use. For self-testing at home. This is a single use device, do not reuse. Not for internal use. Keep out of reach of children. Contains small parts. Do not eat the components of the test. If accidentally swallowed, seek medical attention. Store between 2-30°C (36-86°F). Limit exposing the test to humid conditions, as humidity and high temperature may adversely affect results. Do not freeze. Allow the test to reach room temperature before use, leaving it to sit for at least 30 minutes at room temperature if it is being stored refrigerated. Keep in sealed pouch until use. Do not use if the wrapper is damaged. Do not use past 'use by' date (indicated by the hourglass symbol on the packaging). Use the test immediately after removing it from the pouch. Do not touch the absorbent tip to avoid contamination. Do not immerse, soak or dip the test above the absorbent tip. Use the test immediately after sample collection. Use fresh urine, do not allow urine to sit for a prolonged period at room temperature before using the test. If the urine sample is cloudy, allow it to settle until it is clear. Urine samples may be infectious, use care when handling. Do not use urine containing excessive bacterial contamination. Use a clean, dry collection container for each urine sample. Protect the test from contamination. This test is for human urine only. Don't break up the battery, charge the battery, or throw the battery in fire. Dispose of the test according to local regulations.

Glossary of Symbols

	Consult instructions for use		Tests per Kit		Catalogue #
	Store between 2°C to 30°C		Use by date		Do Not Reuse
	In vitro diagnostic medical device		Lot Number		Manufacturer
	Humidity limitation		Atmospheric pressure limitation		

Contact

If you have questions, reach out to our Technical Support:
support@bttnx.com
1-888-339-9964
Monday-Friday 9:00AM-5:30PM EST.

BTNX, Inc.
722 Rosebank Road,
Pickering, ON L1W 4B2, Canada
Technical Support: 1-888-339-9964

Scan for more resources
www.btnx.com/pregnancy

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