

Rapid Response® At-Home Pregnancy Test

WOMENS HEALTH

The Rapid Response® At-Home Pregnancy Test is a rapid, easy-to-use pregnancy test for the qualitative detection of human chorionic gonadotropin (hCG) in human urine samples. This test is intended for use as an aid in the early detection of pregnancy.



Features

- User-friendly design
- Does not require a urine cup
- Built-in control line
- Detects Human Chorionic Gonadotropin (hCG) in urine

Details


Sample: Urine
Format: Midstream
Read Time: 3 minutes

Storage Condition: 36-86°F/2-30°C
Test Principle: Lateral Flow Assay


Kit Content

- Individually packed test midstream
- Product insert

Testing Procedure

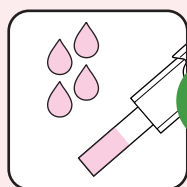

A. Test urine stream

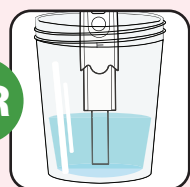


B. Test in collected urine sample.



Read the results at 3 minutes.


Result Interpretation

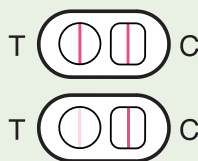

Positive
(Pregnant)



Negative
(Not Pregnant)

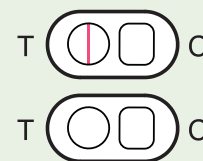

Invalid

For illustrative purposes only. Please read the instructions supplied with the test before use.

Ordering Information

Product Code	Product Name	Contents
HCG-1M1	Rapid Response® At-Home Pregnancy Test	1 Test

To learn more contact your local **Sales Representative**, call us at +1 888 339-9964, or email us at sales@btnx.com