

REF HCG-3C25, HCG-3C50

hCG Test Cassette

Kit Content

- 25/50 Test
- Individually packed test devices
- Package insert



Product Details

Sample: Urine/Serum

Format: Cassette

Time-to-Result: 5 minutes

Storage Condition:

36-86°F/2-30°C


Test Principle: Lateral Flow Assay

Features

- Compact format makes it easy to transport large numbers for tests
- Dual specimen format available
- Internal control line included
- Detects at a concentration of 20 mIU/mL or greater
- Sample can be stored at 36-86°F/2-30°C


Testing Procedure

1

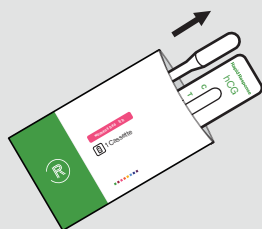

Remove test from sealed pouch.

2

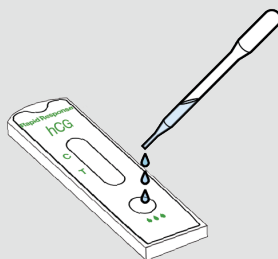

Add 3 drops of urine or serum specimen into well of cassette.

3




5 Minutes

Read results at 5 minutes.


These instructions are for illustrative purposes only. Please read the instructions supplied with the test before use.

Result Interpretation


Positive/Non-Negative

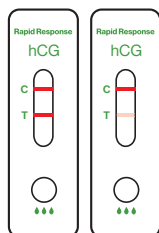

Negative

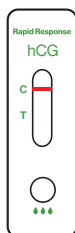

Invalid

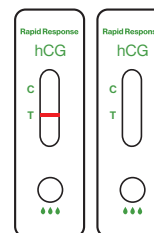

Ordering Information

Product Code	Product Name	Contents
HCG-3C25, HCG-3C50	hCG Pregnancy Test	25/50 Tests


To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at **info@lochnessmedical.com**

REF HCG-3C25, HCG-3C50


hCG Test Cassette

Accuracy

Urine

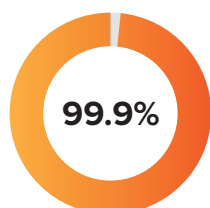


Relative Sensitivity
(97.2%-100.0%)*

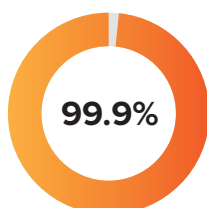


Relative Specificity
(98.0%-100.0%)*

*95% Confidence Interval

Serum

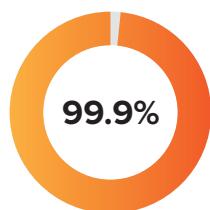

Relative Sensitivity
(97.8%-100.0%)*

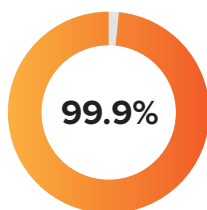

Relative Specificity
(98.5%-100.0%)*

*95% Confidence Interval



Ordering Information

Product Code	Product Name	Contents
HCG-3C25, HCG-3C50	hCG Pregnancy Test	25/50 Tests

To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at **info@lochnessmedical.com**