


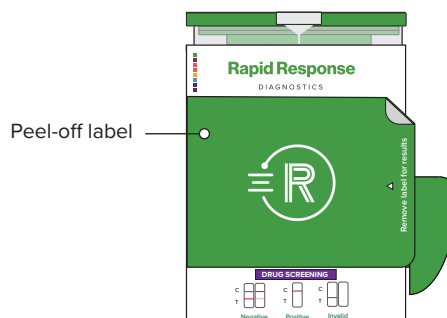


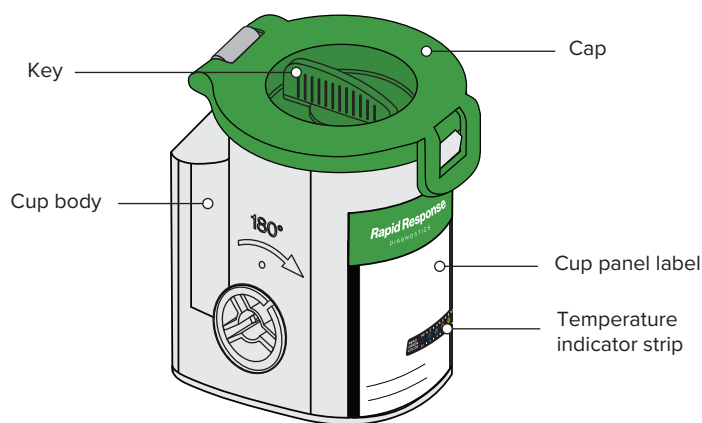


- 1 Remove the cup with its key from the sealed pouch and use it as soon as possible.

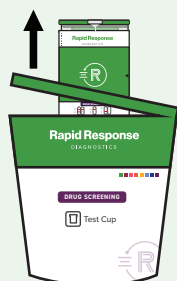


- 2 Donor provides a urine specimen in the cup. If the Operator wishes to run the test at a later time, they should remove the key from the cap and hold on to it. Note: The temperature strip should still be read 2-4 minutes after sample collection.

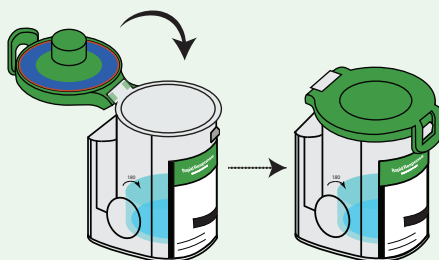


- 3 Donor dates and initials body label and the security seal. The Operator should check the cap for tightness and attach the security seal.

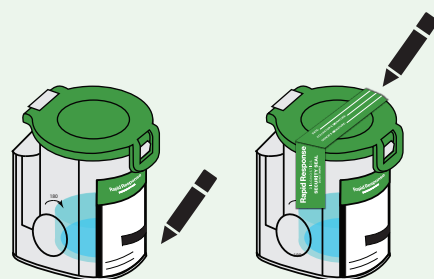

- 4 Donor screws the key clockwise for 180°, and start the timer immediately.

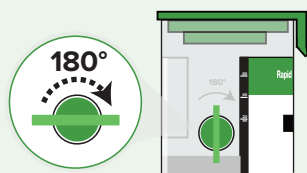

- 5 Remove the peel-off label.

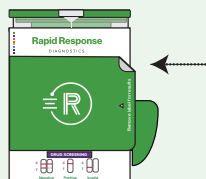

- 6 Check the urine temperature at 2-4 minutes after specimen collection. A green color will appear to indicate the temperature of the urine specimen.




4 Minutes

READ GREEN COLOR	90	2	4	6	8	100
	32	3	4	6	7	38/99

- 7 For the adulteration tests, read results by visually comparing the reacted color blocks on the strip with the color card, and the results should be read at 2 minutes, do not interpret the result after 5 minutes.

	Normal	Abnormal
OXI	Negative	
S.G.	1.003 1.005 1.016 1.025	1.500 1.835
pH	4.0 7.0 9.0	2.0 3.0 10.0 11.0 12.0
NIT	20mg/dl Negative	50mg/dl 100mg/dl
GLUT	Negative	
CREA	100mg/dl 20mg/dl	6mg/dl 10mg/dl

- 8 Drug test results are indicated by the presence or absence of colored band(s) in the result area. The result should be read at 5 minutes. Do not interpret the result after 10 minutes.

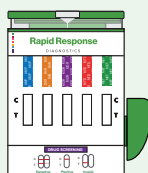

5 Minutes

- 9 Positive test results must be confirmed by another test method. Send the cup and urine specimen intact to a toxicology laboratory for confirmation.



Positive



Negative



Invalid