

REF NTZ-18S9

Rapid Response™ Nitazene Test Strip

(Liquid / Powder)



Nitazene is a highly potent synthetic drug and its prevalence in the illegal drug supply has been on the rise. Evidence suggests nitazenes may be more potent than fentanyl.^{1, 2}

The Rapid Response™ Nitazene Test Strip (Liquid / Powder) is an easy to use, cost-effective dipstick test. It is a rapid, qualitative screening test that quickly and reliably detects Nitazene at cut-off levels above 500 ng/mL. This drug test can provide accurate and fast results as soon as 1 minute. For forensic use only. Not an IVD.

Detects Nitazene Analogues including:

Isotonitazene
Etonitazene
Protonitazene
N-Pyrrolidino Etonitazene

Kit Contents

- Individually packaged test strips with microscopes
- Product insert
- Results interpretation card



University of Dundee Study

The results of the study demonstrate the enhanced sensitivity and specificity of the Rapid Response™ Nitazene Test Strip, with the ability to detect a wider range of nitazene analogues and desnitazenes.

tinyurl.com/mtz69yek



Visit Our Website for more Harm Reduction Resources

To learn more about Rapid Response™ Nitazene Test Strip (Liquid / Powder), scan this QR code.

Lochnessmedical.com/Product/nitazene-test-strips-ntz-18s9

1. Substance Abuse and Mental Health Services Administration. The TEDS Report: Admissions Reporting Benzodiazepine and Narcotic Pain Reliever Abuse at Treatment Entry 2012.
2. JAMA Netw Open. 2023;6(8):e2331264. doi:10.1001/jamanetworkopen.2023.31264


Features

- Small sample requirement of only 5-10 mg
- Detects Nitazene concentrations above 500 ng/mL in liquid / powder samples
- Microscopes included
- Results as soon as **60 seconds**
- **Improved User Guidance:** Every pouch features a QR code linking it to the BTNX Harm Reduction Resource Page - giving every user access to written and video instructions.
- **Results Interpretation:** Results interpretation printed on the back of every pouch and T and C labels printed on each strip to indicate the orientation of the Test and Control lines.

Nitazene Strip



Nitazene Pouch (Back)

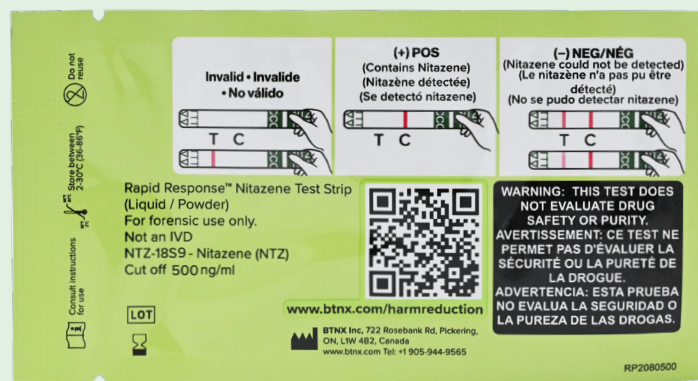


Ordering Information

Product Code	Product Name	Contents
NTZ-18S9-100	Rapid Response® Nitazene Test Strip (Liquid / Powder)	100 Tests
NTZ-18S9-10	Rapid Response® Nitazene Test Strip (Liquid / Powder)	10 Tests

To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at **info@lochnessmedical.com**

Testing Procedure

Step 1 Option 1: Preparing Powder or Pill Samples

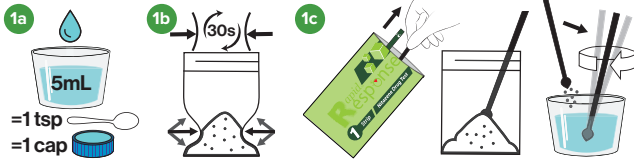

A. Add 5mL of water to a clean container.

B. Mix the drugs thoroughly before testing.


C. Mix the drugs with water. Open the pouch and remove the microscop. Using the microscop, collect one scoop of the powdered drug. Add it to the water. Stir with the microscop until the drug is dissolved in the water.

Step 1 Option 2: Preparing Liquid Drug Residue Samples



A. After the drug shot is prepared and taken into the syringe, the leftover residue can be tested. Add 1mL of clean water to the container or cooker having the residue.

Step 2: Testing

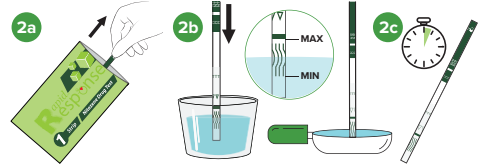

A. Remove the test strip from the pouch.

B. Dip the strip into the solution, up to the wavy lines but not past the solid line, for 10-15 seconds.

C. Remove the strip from the solution and place it on a non-absorbent flat surface. Start the timer. Wait for the colored line(s) to appear.

For illustrative purposes only. Refer to product insert for the full instructions for use.

More from BTNX's Harm Reduction Line Up



Product Code	Details	Quantity
Rapid Response™ Fentanyl Test Strips (Liquid/Powder)		
FYL-18S7-10	Fentanyl Test Strips - 200 ng/mL	10 Tests
FYL-18S7-100		100 Tests
Rapid Response™ Xylazine Test Strips (Liquid/Powder)		
XYL-18S2-10	Xylazine Test Strips - 1000 ng/mL	10 Tests
XYL-18S2-100		100 Tests
Rapid Response™ Benzodiazepine Test Strips (Liquid/Powder)		
BZO-18S3-10	Benzodiazepine Test Strips - 300 ng/mL	10 Tests
BZO-18S3-100		100 Tests
Rapid Response™ Nitazene Test Strips (Liquid/Powder)		
NTZ-18S9-10	Nitazene Test Strips - 500 ng/mL	10 Tests
NTZ-18S9-100		100 Tests
NTZ-18S26-10	Nitazene Test Strips - 2000 ng/mL	10 Tests
NTZ-18S26-100		100 Tests
Rapid Response™ Medetomidine Test Strips (Liquid/Powder)		
MED-18S2-10	Medetomidine Test Strips - 1000 ng/mL	10 Tests
MED-18S2-100		100 Tests
Rapid Response™ Drug Checking Test Kit (Liquid/Powder)		
BZO-18S3-5K	Benzodiazepine Test Strips - 300 ng/mL. Includes buffer tubes, swabs, and tube stand.	5 Tests
FYL-18S7-5	Fentanyl Test Strips - 200 ng/mL. Includes buffer tubes, swabs, and tube stand.	
MED-18S2-5K	Medetomidine Test Strips - 1000 ng/mL. Includes buffer tubes, swabs, and tube stand.	
NTZ-18S9-5K	Nitazene Test Strips - 500 ng/mL. Includes buffer tubes, swabs, and tube stand.	
NTZ-18S26-5K	Nitazene Test Strips - 2000 ng/mL. Includes buffer tubes, swabs, and tube stand.	
XYL-18S2-5K	Xylazine Test Strips - 1000 ng/mL. Includes buffer tubes, swabs, and tube stand.	

To learn more contact your local **Sales Representative**, call us at **+1 888 506-2658**, or email us at info@lochnessmedical.com



**Visit Our Harm Reduction
Resource Page**

lochnessmedical.com/category/harm-reduction