



Rapid Response™ Spiked Drink Testing



Scan to learn more
[lochnessmedical.com/
subcategory/spiked-drink-testing](https://lochnessmedical.com/subcategory/spiked-drink-testing)

Test drinks for **Rohypnol (Benzos), Ketamine, and GHB.**

In social settings where beverage tampering or spiking is a concern, testing your drink is a defensive step to protect against potential harm.



Stay Safe, Carry a Test

Test your drink and keep it handy, knowing you can check for safety on the spot.



Quick Results


Get a peace of mind in minutes! No more waiting around. Perfect for bars, festivals, and any social event where safety matters.



Compact Confidence

Easy to carry on-the-go, so you're always prepared wherever the night takes you.

Rapid Response™ Spiked Drink Test

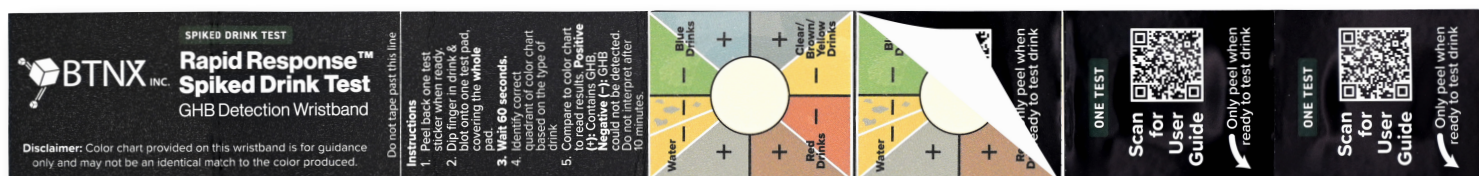

A wearable wristband with patches covering large sample pads.

HOW TO USE

Dip your finger in your drink and blot it on the test.

PACKAGING

10 wristbands, 4 tests per wristband.

TEST PRINCIPLE

Color-changing pad

DETECTS

GHB

RESULTS IN

60 sec

RESULTS INTERPRETATION

GHB DETECTION WRISTBAND

POSITIVE



Clear/Brown/
Yellow Drinks



Water



Blue Drinks



Red Drinks

NEGATIVE



Water + Clear
/Brown/Yellow



Water




Blue Drinks



Red Drinks


Rapid Response™ 3-in-1 Spiked Drink Pen

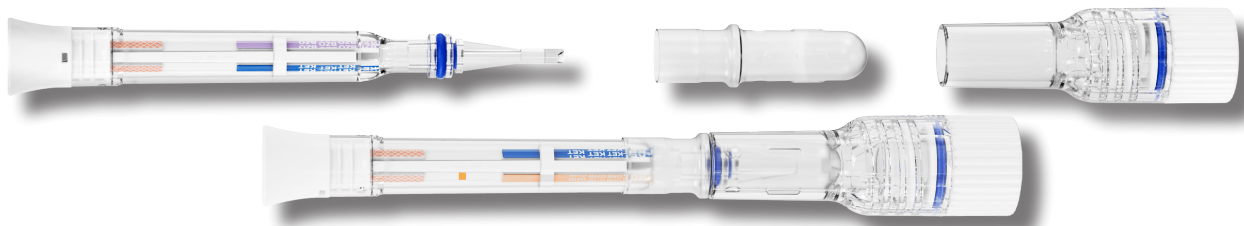

RESULTS INTERPRETATION


A dipstick test with built-in test strips and buffer.

BZO/KET TEST

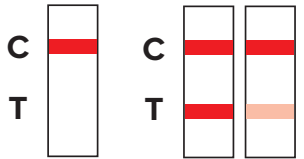

POSITIVE NEGATIVE

GHB TEST

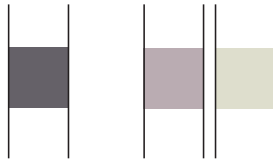

POSITIVE NEGATIVE

HOW TO USE

Dip the test pen in the drink, remove, and wait for results.

TEST PRINCIPLE

Lateral flow/Color lines, Color-changing pad

RESULTS IN

3 min (GHB)
5 min (BZO/KET)

PACKAGING

1, 3, 5, or 25 test pen(s)

DETECTS


GHB, Benzodiazepines, Ketamine

Rapid Response™ 3-in-1 Spiked Drink Drug Test Kit

A 3 piece test kit with a test cassette featuring built-in test strips.


RESULTS INTERPRETATION

BZO/KET TEST

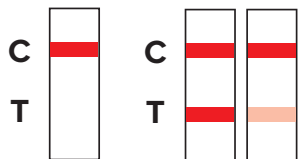

POSITIVE NEGATIVE

GHB TEST

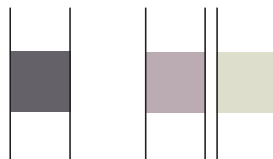

POSITIVE NEGATIVE

HOW TO USE

Pipette a sample of the drink into the buffer tube, then add drops to the test.

TEST PRINCIPLE

Lateral flow/Color lines, Color-changing pad

RESULTS IN

5 min

PACKAGING

10 test cassettes, each comes with a pipette and a buffer tube.

DETECTS

GHB, Benzodiazepines, Ketamine



ORDERING INFORMATION

Product Code	Rapid Response™ Product	Qty
GHB-8WB158-10	Spiked Drink Test (Wristband)	10T
D3.39-8C29-10	3-in-1 Spiked Drink Drug Test Kit	10T
D3.39-8DH	3-in-1 Spiked Drink Pen	1/3/5/25T



Contact Us:

Tel: +1888-506-2658

Email: orders@lochnessmedical.com

DISCLAIMER

For beverage drug testing use only. Not an IVD.

This test is NOT intended to determine the purity, composition, or if the liquid being examined is safe to use. A positive or negative test result is NOT a guarantee that the liquid being tested is safe or unsafe to use. Any tested liquid is consumed or used at the consumer's own risk.

Rapid Response™ Spiked Drink Tests provides information to consumers but does not eliminate risks. Users of Rapid Response™ Spiked Drink Test, 3-in-1 Spiked Drink Pen, and 3-in-1 Spiked Drink Drug Test Kit accept responsibility for any consequences, including injury or death, that may occur after consuming a tested liquid.

Many factors come into play when testing a liquid, including but not limited to mixture of multiple substances, solubility, and pH of the sample.

Technical or procedural errors, as well as other substances, and factors may affect the accuracy of the Rapid Response™ Spiked Drink Test, 3-in-1 Spiked Drink Pen, and 3-in-1 Spiked Drink Drug Test Kit, and can cause false results.

A positive result indicates the presence of GHB, Ketamine, and/or Benzodiazepine only and does not indicate quantity or concentration.

A negative result does not rule out the presence of GHB, Ketamine, and/or Benzodiazepine, or any other potentially harmful substances.