# INSTRUCTIONS (ENGLISH)

## Rapid Response

## Spiked Drink Test GHB Detection Card Instruction Card

REF GHB-8CD158

The colors of the illustration do not represent the colors on the actual card. Please refer to the physical card for accurate colors.


1 Read the instructions.  
Peel off one test sticker when ready.

2 Dip finger in drink & blot onto the test pad.  
Saturate the whole pad.

3 Wait 60 seconds.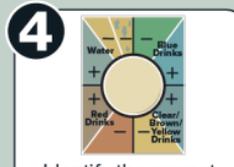

4 Identify the correct quadrant of the color chart based on your type of drink.

**Note for even more discreet testing:** Without removing the protective stickers carefully cut the band between test to create individual test squares.

**BTNX Inc.**  
Tel: +1 888-339-9964  
Email: support@btnx.com  
www.btnx.com  
Mon.-Fri. 9:00 AM - 5:30 PM EST

Scan for More Resources

www.btnx.com/GHBCard

## Result Interpretation

Compare the color of the test pad to the quadrant that corresponds to the color of your drink. If the color matches the part of the quadrant with the + it is positive, if it matches the part of the quadrant with the - it is negative. Compare to outer color chart on the card to read results.

Do not interpret after 10 minutes

### NEGATIVE

-  
Could not detect GHB.

### POSITIVE

+  
Contains GHB.

### INVALID

If the color pad is green or blue *before* applying the sample, it indicates procedural errors.  
Do not use the test.

Mineral content of water may cause blue spots on a *mostly* yellow result; this does not indicate a positive result.

- This test is NOT intended to determine the purity, composition, or if the liquid being examined is safe to use. A positive or negative test result is NOT a guarantee that the liquid being tested is safe or unsafe to use. Any tested liquid is consumed or used at the consumer's own risk.
- The sample pad should be yellow when first opened. Do not use the test if the sample pad is green or blue prior to adding sample.
- Humidity and sunlight may affect the results, always use the sample pad immediately after peeling the protective sticker. Do not reuse the test.
- Test should be stored at 36°F to 86°F (2°C - 30°C).
- For Beverage Drug Testing use only. Not an IVD.

**BTNX Inc.**  
722 Rosebank Road,  
Pickering, ON L1W 4B2 Canada

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