Rapid Response®

Total Cholesterol At-Home Tracking Test (Whole Blood)

REF TCH-11CDACH1

Product Insert


SCAN FOR MORE RESOURCES

www.btnx.com/product/total-cholesterol-home-tracking-test

A test card for the Semi-quantitative detection of Total Cholesterol in human whole blood. For self-testing. For *in vitro* diagnostic use only.

Materials Provided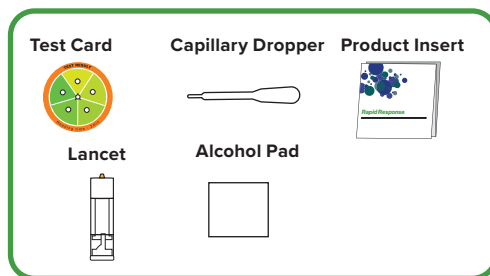

Materials required but not provided: Timer.

Specimen Collection and Preparation

- Acceptable specimens include capillary whole blood from the fingertip.
- Whole blood specimens should be tested immediately after lancing.
- Fingertip sample must be obtained from a free spontaneous blood flow after the puncture.
- Do not collect capillary blood by pressing or squeezing the finger too hard** as this will push fluid from the tissue into the blood and may cause false results.


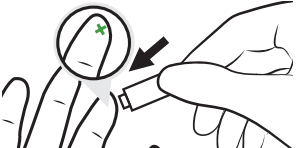
Test Procedure

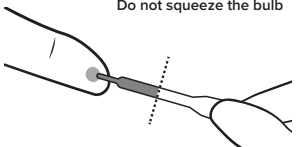
Wash your hands with soap and rinse with clear warm water.

Bring the test pouch to room temperature before opening it. Open the pouch, remove the test card and place it on a clean, level surface. Run the test within one hour; best results will be obtained if the test is performed immediately after opening the foil pouch.

Remove the dropper, lancet and alcohol pad, place them close to the test card.

Test Procedure
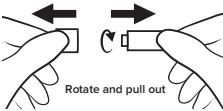
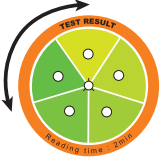
- Carefully pull off and dispose the released cap of the lancet. 
- Use the provided alcohol pad to clean the fingertip of the middle or ring finger as the puncture site. Allow the finger to air dry. 
- Press the lancet on the side from where the cap was extracted. The tip retracts automatically and safely after use. Massage the hand without touching the puncture site by massaging the hand towards the fingertip of the middle or ring finger to be punctured. 

- Keeping the hand down massage the end that was pricked to obtain a blood drop. 
- Without squeezing the dropper bulb, put it in contact with the blood. The blood migrates into the to the line indicated on the capillary dropper. You may massage again your finger to obtain more blood if the blood does not reach the indicated line. Avoid of air bubbles. 

- Release the blood collected into the sample well of the card by squeezing the dropper bulb. 

- Wait for the blood to be totally dispensed in the well and wait 2 minutes before turning the test kit over. **DO NOT turn over the test kit before 2 minutes.** 

Test Procedure

- Turn over the test kit after 2 minutes. 
- Read the results by rotating the card and matching the closest matching color within 30 seconds. Do not read the result **after 2 minutes 30 seconds** of applying the blood sample. 

Interpretation of Results

< 200mg/dL (< 5.2mmol/L)	Desirable
200-249mg/dL (5.2-6.5mmol/L)	Borderline High
≥ 250mg/dL (≥ 6.5mmol/L)	High

Intended Use

The Rapid Response® Total Cholesterol At-Home Tracking Test is a single-use device that utilizes the dry chemistry method for the semi-quantitative detection of total cholesterol in human whole blood. The test provides a preliminary analytical result. Clinical considerations and professional judgment should always be applied when interpreting the results of this test.

Summary

Cholesterol is a lipid that plays an essential role in cell membranes, hormone production, and vitamin D synthesis. Elevated cholesterol levels are a known risk factor for cardiovascular disease. Measuring total cholesterol in blood provides useful information for monitoring and managing personal health.

Principle

A blood sample is applied to the sample area of the test card, where it infiltrates evenly and rapidly. During this process, blood cells are filtered and plasma diffuses into the reaction layer. Enzymes and chemical reagents in the reaction layer produce a color change. The intensity of the color is compared with the provided color chart to estimate the total cholesterol level.

Reagents

Reagent concentrations are based on dry weight at the time of impregnation and may vary within normal manufacturing tolerances.

Reagent	Composition
Total Cholesterol (TC)	Cholesterol Esterase > 0.3U
	Cholesterol Oxidase > 0.16U
	4-Aminoantipyrine > 0.06mg

Precautions

- For self-testing, *in vitro* diagnostic use only.
- The test card should remain in the closed packaging bag until use.
- Do not use the test after the expiration date.
- Do not touch the sample area of the card.
- Discard any discolored or damaged card.
- Dispose of the used test card according to local regulations.

Storage and Stability

- Store the test card as packaged in its sealed pouch at room temperature or under refrigeration (2-30°C; 35.6-86°F). Keep the test away from direct sunlight and **DO NOT FREEZE**. The test card is stable until the expiration date printed on the package and should not be used beyond that date. **NOTE:** stability may be reduced under conditions of high humidity.

Limitations of Procedure

No significant effect was observed at the following concentration for each substance:

Ascorbic acid: 3 mg/dL	Urea: 500 mg/dL
Bilirubin 20 mg/dL	EDTA: 640 mg/dL
Creatinine: 30 mg/dL	Heparin-Lithium: 100 mg/dL
Hemoglobin: 400 mg/dL	Hematocrit: Between 30~55% has no effect on test results.

Performance Characteristics

Repeatability/Reproducibility

A study was conducted using samples containing known cholesterol values. Each sample was tested ten times with the Rapid Response® Total Cholesterol At-Home Tracking Test. The agreement of results was above 93%.

Accuracy

A clinical study was conducted comparing the Rapid Response® Total Cholesterol At-Home Tracking Test with the reference method. This clinical study was performed using 80 venous blood samples. Comparison results between professional and laymen revealed a very high concordance of 95%.

Bibliography

- Henry, J. B. Clinical Diagnosis and Management by Laboratory Methods. (2001) 24:479-481
- Dacie and Lewis, Practical Haematology, Tenth Edition, 2006 Index of Symbols

Glossary of Symbols

Consult instructions for use or consult electronic instructions for use	Contains sufficient for <n> tests	Caution
Temperature limit	Use by date	Catalogue number
<i>In vitro</i> diagnostic medical device	Batch code	Do Not Reuse
Do not use if package is damaged and consult instructions for use	Manufacturer	

BTNX, Inc.
722 Rosebank Road
Pickering, ON L1W 4B2 Canada
Technical Support: 1-888-339-9964
www.btnx.com
Monday-Friday 9:00AM - 5:30PM EST

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